

15mm

ETIL PROFESSIONAL

ALCOL TEST SALIVA CE

1 Saturate the reactive pad with saliva from collection cup or by applying saliva directly to the pad. (It usually takes 6-8 seconds to be saturated)

2 Read the result after 2/3 minutes: compare the color of the pad with the scale of the results on the back of the package.


SINGLE USE TEST ②
TEST IVD **IVD**
ONLY FOR PROFESSIONAL USE

INDICATIVE QUALITY TEST


This Test provides only a preliminary analysis
The user is solely responsible for deciding to drive.

15mm


RESULTS



0.00




0.20



0.50


- Read the instructions before performing the test
- Before testing, take it to a temperature between 15°C and 30°C
- Do not read the results after 10 minutes

- Abstain from placing anything in the mouth prior to beginning the test. (avoid food, drinks, tobacco, chewing-gum)
- Store between 2°C and 30°C
- Keep the test out of reach of children
- Avoid release container to the environment



SCREEN ITALIA S.r.l. Via Volumnia, 40/b
06135 Ponte San Giovanni - Perugia - Italia
www.screenitalia.it info@screenitalia.it

LOT



ScreenItalia

Test per la tua salute